

Asirox®
EVOLVE



ALL PHOTOTYPES,
ALL YEAR ROUND

ABOUT US

Much like Lynton, Asirox® is known for creating pioneering medical devices, which are the culmination of years of experience and expertise.

As the UK's leading manufacturer of IPL and laser technology, our partnership with Asirox® is a natural continuation of our long-standing commitment to supplying our customers with outstanding products that meet their needs.


We are Asirox®'s exclusive distribution partner in the UK and we will be working closely with them to ensure that our existing and prospective customers are acutely aware of just how reliable, effective and impressive the EVOLVE system truly is.

The entire Lynton team are looking forward to introducing the future of pain-free diode hair removal treatments to the UK market.

We comply with the standards required by certificate ISO 13485/2016 of Quality Management System.

Asirox®
EVOLVE

EVOLVE	
Type of Laser	Diode Laser
Wavelength	800-810nm
Spot Size	10x10mm
Fluence	1-40 J/cm²
Pulse Duration	10-100 ms
Work Frequency	Standard Mode: 1-4 Hz Motion Mode: 5 and 10 Hz
Cooling System	Water
Power Supply	90-250v

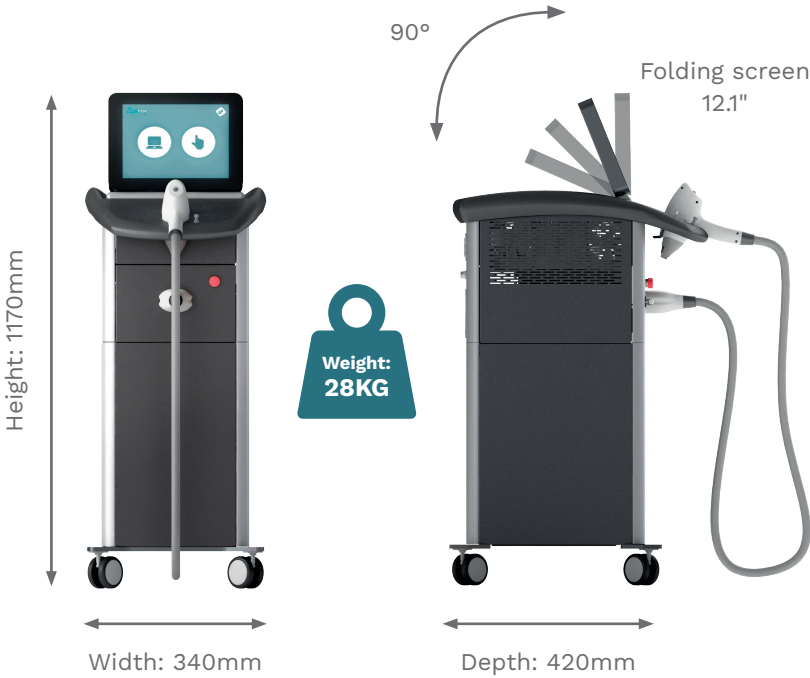

Diagonal Medical Eshtetic SL is in compliance with the applicable requirements of 93/42/EEC Directive as amended, Annex II without point 4, transposed in Italy by Dlgs. 46 of 1997/02/24 as amended Medical Devices of Laser device for treatment of pseudofolliculitis and hirsutism.

DIAGONAL MEDICAL ESTHETIC S.L.
C/ SICILIA 410 A
08025 (Barcelona)
Made in Spain



For more information:
Tel: +44 (0)1477 536 977
info@lynton.co.uk
lynton.co.uk



Lynton
Exclusive distributor
in the UK for
Asirox®

DIODE HAIR REMOVAL -
THE FUTURE IS NOW

lynton.co.uk



ASIROX® EVOLVE



Asirox® EVOLVE is a medical device created for hair reduction on all skin phototypes (Fitzpatrick I-VI), as well as treatment for pseudofolliculitis and hirsutism.

Discover its innovative work systems and enjoy its benefits from the first pulse.




Mode Standard

Our ePulse Technology® provides a 90° ramp and a totally square pulse to obtain the necessary energy to remove hair with one single pulse. The 4Hz speed can be used in all parameters, ensuring fast, high-quality treatment.

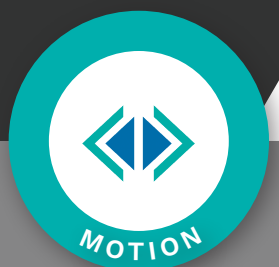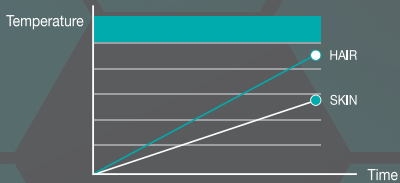

Mode Motion

Energy is gradually applied to the skin as the temperature increases. As the skin's thermal relaxation time (TRT) is shorter than that of the hair on the treatment area, there is no associated pain. It can also be used on all phototypes throughout the year. This mode is incredibly versatile thanks to the speed selection option (5Hz-10Hz). It allows you to work on both large and small areas with ease.

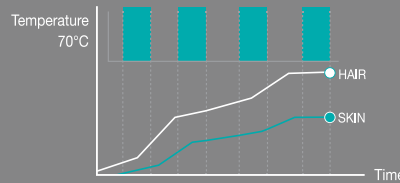

Mode Expert

You can alter established parameters (fluence, pulse duration and repetition rate) to make personalised treatments according to each patient's needs. All used parameters are always visible during treatment.

SATISFIED PATIENTS

Pain-Free Treatment

Comfortable treatment thanks to Dual Motion and Icetip Technology.

Icetip Technology

Bring the tip temperature down to -8°C for safer treatments.

Effective And Safe

Suitable for tanned skin and darker phototypes, whilst maintaining excellent results.

BUSINESS BENEFITS

High Demand Treatment

Each day, more people from different social and ethnic backgrounds look for hair reduction treatments, with all of them looking for an effective, fast and comfortable option.

Stepping Stone In Your Clinic

For patients, laser hair removal is an excellent stepping stone into the world of Aesthetic Medicine.

Fast Treatment

Asirox® EVOLVE works at a speed of 4cm²/second to cover an area of 240 cm² in 1 minute.

BE DIFFERENT AND EVOLVE YOUR BUSINESS

“

TESTIMONIALS



Dr. Sofija Malbaša
Laser Centar - Serbia

“I am very satisfied with the Asirox® EVOLVE's two working modes. The ability to use Standard or Motion and to change automatic parameterisation according to my patient's needs allows me to provide excellent results from the first session.”



Dr. Elena Kneštiapina
Ten Zen - Barcelona

“Hair removal with the Asirox® EVOLVE is extremely fast, comfortable and it can be used all year round and on all types of skin. Thanks to this, traffic in my clinic has increased and I have achieved an impressive ROI with this device.”



Dr. Jorge Baños
Clínica Zaneo-
Institute-Mexico

“What I love about this technology is the versatility to treat different phototypes for hair removal and how easy it is to programme EVOLVE and perform treatments. In our clinic, it has allowed us to help a very wide range of patients and to treat them in a very fast, effective and extremely safe way.”



Dr. Sam Hills
Lynton Lasers-UK

“The Asirox® EVOLVE System is a great addition to any clinic looking to introduce laser hair removal. Clients love having the option of pain-free hair removal! The intuitive user interface and ergonomic handpiece make it a pleasure to use. We are very satisfied with the clinical results of EVOLVE.”

BEFORE AND AFTER



Dr. Sofija Malbaša - Laser Centar

Results on a female chin after 1 session of Asirox® EVOLVE treatment



Dr. Jorge Baños - Clínica Zaneo

Results on a female armpit after 3 sessions of Asirox® EVOLVE treatment



Dr. Sam Hills - Lynton Lasers

Results on a male back after 4 sessions of Asirox® EVOLVE treatment.

